



OPASTER'ANIOS

TESTING STRIPS - TESTNE TRAKE OPASTER'ANIOS

INDICATIONS

50 specific reactive strips for testing the concentration of orthophthaldehyde in the solution OPASTER'ANIOS.

INSTRUCTIONS FOR USE

- Use the testing strip at the start of a session or when restarting after an interruption.
- Soak the strip for one second.
- Shake off excess liquid.
- Read the result 60 seconds after steeping. Respect this time before reading.

STORAGE CONDITIONS

Keep the flask firmly closed at a temperature between +4°C and +30°C, in a dry area for 24 months. Store away from light and moisture.

If these conditions are respected, testing strips are usable until 90 days after opening.


PRECAUTIONS FOR USE

Dangerous – respect the precautions for use. Class I medical device. Before every use, read the label and information concerning the product.


CONFORMITY/ SUKLADNOST



Strip before utilisation
Traka prije uporabe

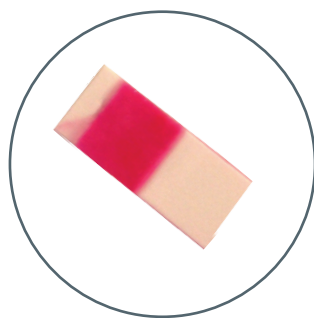

Pink uniform CONFORM
Ujednačena roza boja ODGOVARA

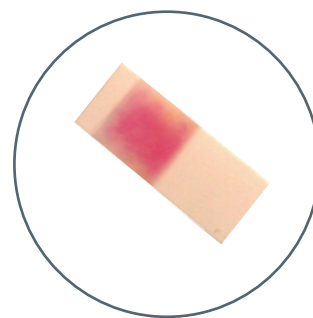

Pink not uniform NO CONFORM
Neujednačena roza boja NE ODGOVARA

INDIKACIJE

50 posebnih reaktivnih traka za ispitivanje koncentracije ortoftaldehida u otopini OPASTER'ANIOS.

UPUTE ZA UPORABU

- Koristiti test traku na početku dezinfekcije ili prilikom nastavka aktivnosti nakon prekida
- Traku natapajte 1 sekundu.
- Otrесite višak tekućine.
- Očitajte rezultat 60 sekundi nakon natapanja. Poštujte zadano vrijeme prije očitavanja.

UVJETI ČUVANJA

Pakiranje čuvajte čvrsto zatvoreno, na temperaturi između +4 °C i +30 °C, na suhom mjestu 24 mjeseca. Zaštite ga od svjetlosti i vlage.

Ukoliko se ovi uvjeti poštuju, testne trake iskoristive su u razdoblju od 90 dana nakon otvaranja.

MJERE OPREZA PRI UPORABI

Opasno – poštujujte mjere opreza pri uporabi Medicinski uređaj klase I. Prije svake uporabe pročitajte deklaraciju i informacije o proizvodu.



PLIVA HRVATSKA d.o.o.,
Zastupnik i distributer za Republiku Hrvatsku,
Prilaz baruna Filipovića 25,
10000 Zagreb, Hrvatska.
Tel.: + 385 1 37 20 000,
Faks: + 385 1 37 20 111;
www.pliva.hr, www.plivamed.net



Pavé du Moulin
59260 Lille-Hellemmes - France
Tel. +33 3 20 67 67 67 - Fax : +33 3 20 67 67 68
www.anios.com

SAMO ZA ZDRAVSTVENE RADNIKE

HR100110-180523- Non contractual
photographies - Izvanugovorne fotografije
HR/ANIO/17/0010 - datum sastavljanja: 04/2017